

The Bionix Carbon Fiber 45 Degree Attachment is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.


Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

COMPONENTS

A:: Suprone Plus

B:: CF 45-degree Attachment

ASSEMBLY DIAGRAM

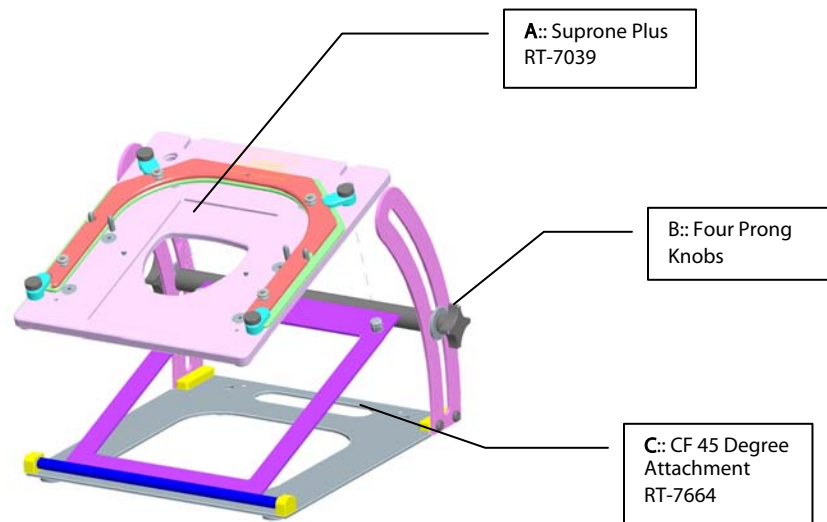

OPERATING INSTRUCTIONS

1. Position the Bionix Carbon Fiber 45 Degree Attachment (C) for the Suprone Plus (A) on the treatment couch as desired using the Bionix SecureFit Bar (SecureFit Bar Part #'s RT-5010, RT-7010- not shown).
2. Attach the BIONIX Suprone Plus to the Attachment (A) as shown in the Assembly Diagram above.
Note: Be sure the Suprone Plus is securely pressed into place.
3. Place the desired head support onto the Bionix Carbon Fiber 45 Degree Attachment (C) for the Suprone Plus (A) for normal treatment or select the prone cushion for prone treatment.
4. By turning the knobs counterclockwise, loosen the black four prong knobs (B) enough that the board moves freely; select the desired position by aligning the angle indicator mark to the desired angle, as shown in figure 1; then tighten the knobs.
5. Select a thermoplastic frame and prepare as instructed.

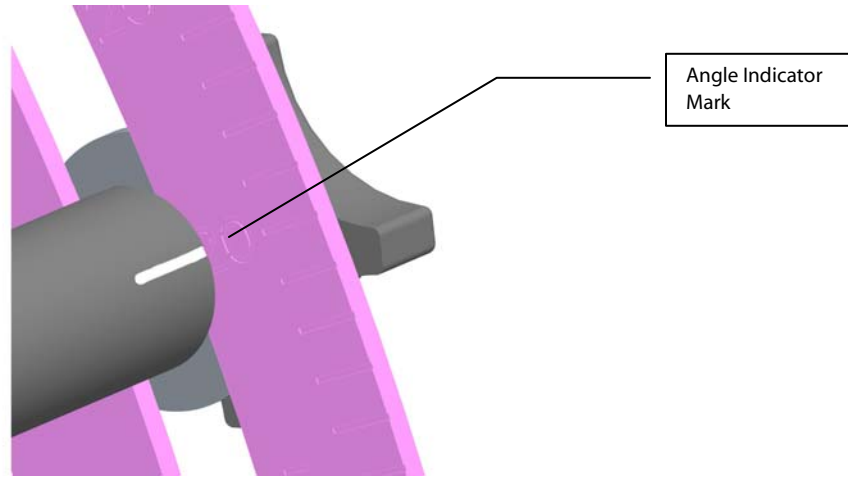


Figure 1

6. Center the Low Melt Thermoplastic Frame directly over the patients chin and draw the mask over the face and push down towards the locating pins in the head region.
7. Position and tighten the thermoplastic frame clamps to the locked position, making sure the Low Melt Thermoplastic Frame is secured in place.
8. Form the Low Melt Thermoplastic Mask to patient's facial contours.

Note: *Thermoplastic should be cool to touch and returned to its original color before removing it from the patient. If removed too soon it will cause shrinkage.*

CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

Note: Do not use alcohol based cleaners on this product.

WARRANTY

The Bionix Carbon Fiber 45 Degree Attachment for the Suprone Plus is warranted to be free from defects in workmanship and materials, under normal use, for a period of 3 years from the date of original purchase from Bionix. Bionix' sole obligation under this warranty shall be, at Bionix' option, to replace the product or part with a comparable product or part, repair the product or part, or if neither repair nor replacement is reasonably available, Bionix may, in its sole discretion, refund to the customer the purchase price paid for the product or part.

Bionix Radiation Therapy
5154 Enterprise Blvd.
Fax: 800-455-5678

Toll Free: 800-624-6649
Toledo, OH 43612
www.BionixRT.com