

The Bionix Comfort Hold Thigh Positioner is intended to be used for the positioning and re-positioning of patients undergoing treatments or imaging.

**Caution:** Federal law restricts this device to sale by or on the order of a licensed physician.


## COMPONENTS

A: Thigh Cushion Base  
B: Thigh Cushion

C: Thigh Cushion Riser

## ASSEMBLY DIAGRAM

Diagram coloring used to demonstrate component differentiation. May not be actual product color.

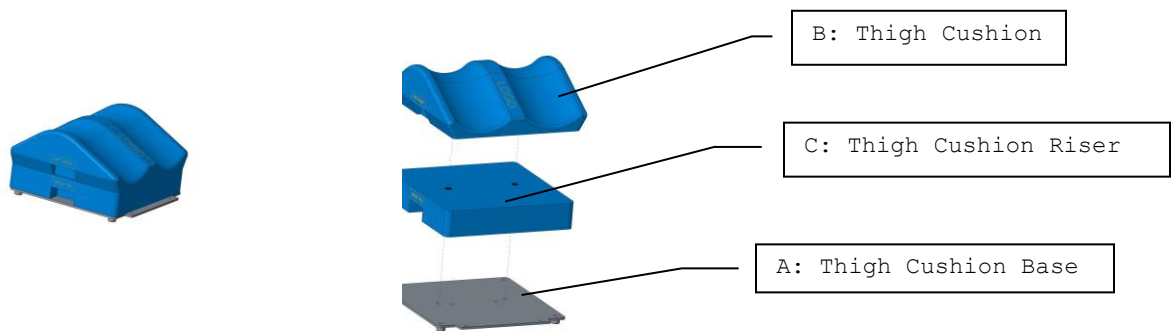

## OPERATING INSTRUCTIONS

1. In order to begin using the Bionix Comfort Hold Thigh Positioner, you will need to have a SecureFit Bar (not shown) which will need to be secured to the treatment couch.
2. Place the Thigh Cushion Base (A) on to the SecureFit Bar (not shown).
3. Continue by assembling the pad set as shown in the Assembly Diagram.
  - a. Thigh Cushion Riser (C) on to the Thigh Cushion Base (A). The Thigh Cushion Riser is optional for use depending on patient size, comfort and optimum positioning.
  - b. Thigh Cushion (B) on to the Thigh Cushion Base (A) or the Thigh Cushion Riser (C)

## CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

**Notice: Do not use alcohol based cleaners on this product**

## WARRANTY

The Comfort Hold Thigh Positioner is warranted to be free from defects in workmanship and materials, under normal use, for a period of 3 years from the date of original purchase from Bionix. The sole obligation of Bionix under this warranty shall be to replace or repair the product or part with a comparable product or part. To register your product visit the following website: [http://www.bionixrt.com/RT\\_Surveys/WarrantyForm.aspx](http://www.bionixrt.com/RT_Surveys/WarrantyForm.aspx)

Bionix Radiation Therapy  
5154 Enterprise Blvd.  
Toledo, OH 43612  
Toll Free: 800-624-6649  
Fax: 800-455-5678  
[www.BionixRT.com](http://www.BionixRT.com)



Obelis s.a • Boulevard Général Wahis 53  
1030 Brussels, BELGIUM • Tel: +(32) 2.732.59.54  
Fax: +(32) 2.732.60.03 • E-Mail: mail@obelis.net



Bionix Development Corp. • 5154 Enterprise Blvd.  
Toledo, OH, 43612 • U.S.A. • Phone: 800-551-7096  
Fax: 800-455-5678 • www.Bionix.com

