

The Bionix Pelvis Belly System Plus is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.

Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

## COMPONENTS

A: Treatment Base


B: Adjustable Face Cushion

C: Small and Medium Stomach Cushions

(Large size is optional)

E: Leg Cushion


## ASSEMBLY DIAGRAM

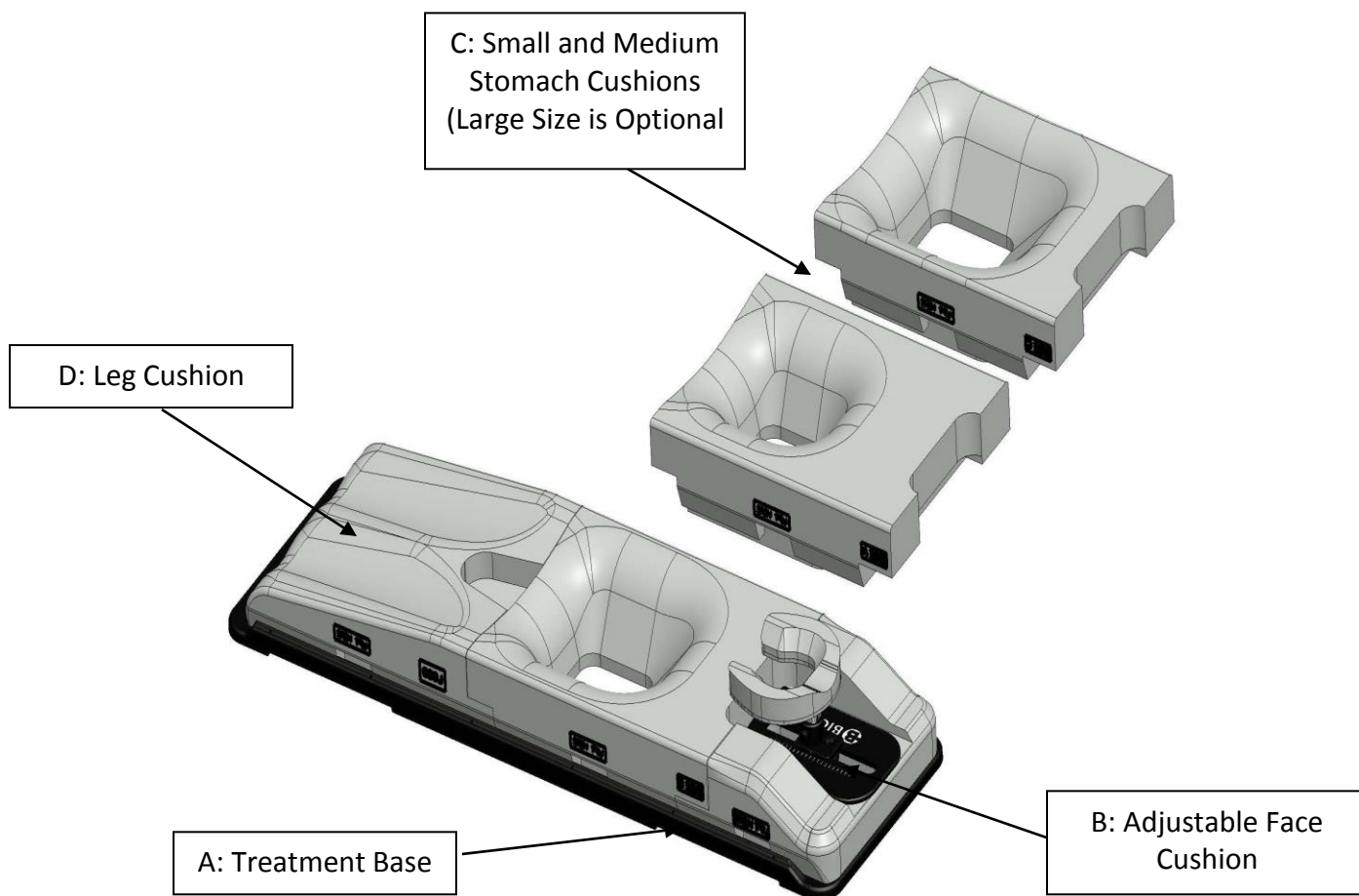

**Figure 1**

## OPERATING INSTRUCTIONS

1. **Secure the treatment base onto the couchttop with the appropriate SecureFit bars.** Bionix advises using two SecureFit bars (which are not included with this system) for the treatment base. Note that the engraved Bionix “B” indicates the top of the treatment base.
2. Attach the leg cushion on the treatment table at the desired position, as shown in **Figure 1**.
3. Select and place the desired pads on the Treatment Base as shown in **Figure 1**.
4. **Place the face cushion on the device.** Move the adjustable head rest to the appropriate position favored by the patient. (See Figure 2 below) Be sure to note what position is being used on the patient positioning checklist. Loosen the knobs and press the button to adjust the face cushion up and down. Secure with the knob and adjust to the patient’s preferences.

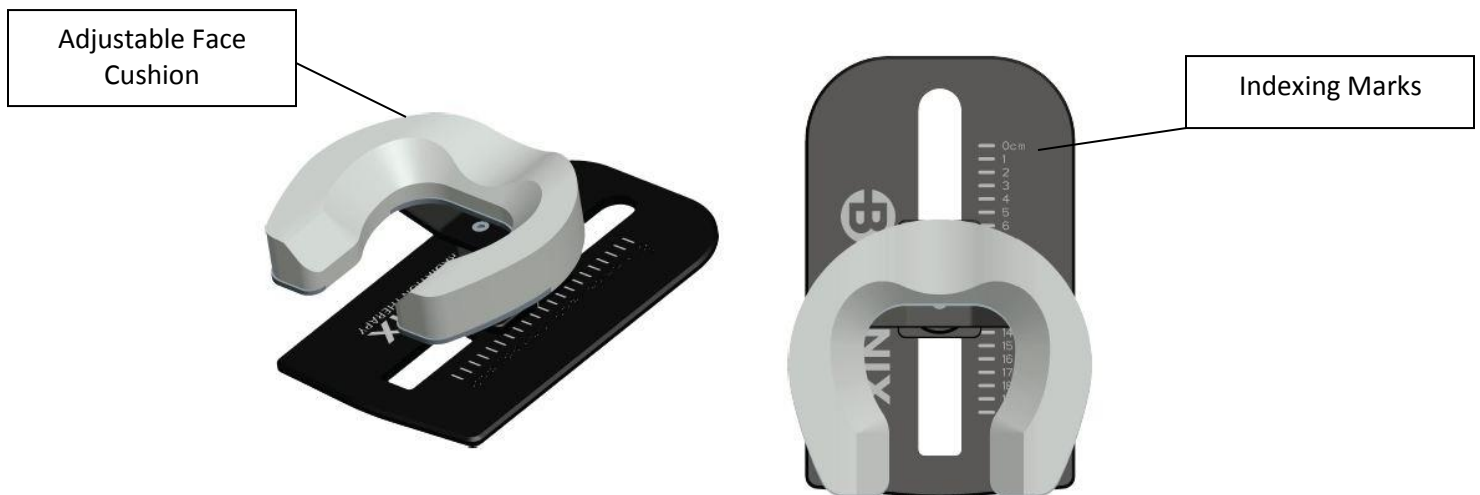

**Figure 2**

5. **Position the patient on the Pelvis Belly System in the prone position**, with the crest of the pelvic bone tilting towards the cutout in the stomach pad. An optional ankle cushion can also be used for a more comfortable setup



## OPTIONAL ACCESSORIES

DESCRIPTION	PART NUMBER
Pelvis Belly Pad Set (with ankle cushion)	RT-5105-10
Large Abdominal Insert	PBRT-5106

## CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water-based antiseptic cleaner or foam. Allow to dry before next patient use.

Notice: Do not use alcohol-based cleaners on this product.

## WARRANTY

The Pelvis Belly System Plus is warranted to be free from defects in workmanship and materials, under normal use, for a period of 3 years from the date of original purchase from Bionix. The sole obligation of Bionix under this warranty shall be to replace or repair the product or part with a comparable product or part. To register your product visit the following website:

[http://www.bionixrt.com/RT\\_Surveys/WarrantyForm.aspx](http://www.bionixrt.com/RT_Surveys/WarrantyForm.aspx)

Bionix Radiation Therapy  
5154 Enterprise Blvd.  
Toledo, OH 43612  
Toll Free: 800-624-6649  
Fax: 800-455-5678  
BionixRT.com