


The Bionix Butterfly Breast Board- Timo & Silverman is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.

Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

COMPONENTS

- A:: O-Ring
- B:: Butterfly Breast Board
- C:: Hand Posts
- D:: Attachment Peg

ASSEMBLY DIAGRAM

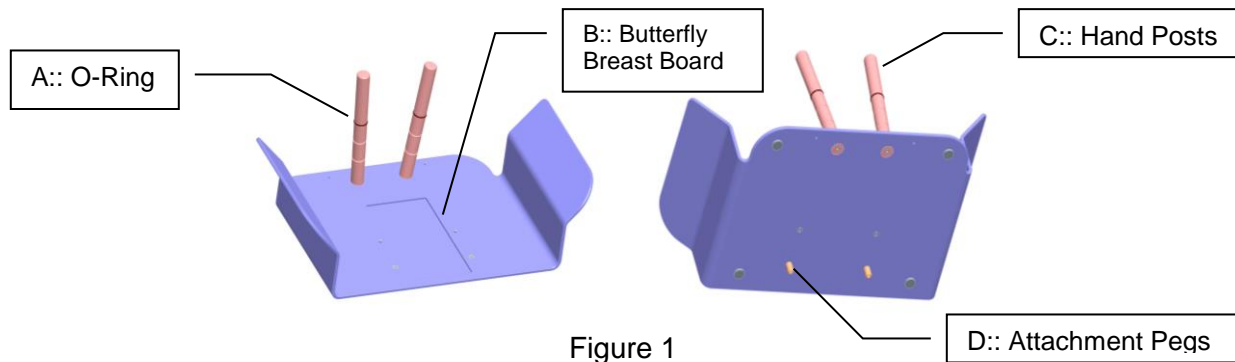

NOTE: The Silverman Head Cups must be punched to be used with the Locating Screws

OPERATING INSTRUCTIONS

1. Position the Butterfly Breast Board on the treatment table.
2. Position Timo or Punched Silverman head cup on Butterfly Breast Board.
3. Position the patient on the Butterfly Breast Board.
4. Move the O-rings on the hand posts to the desired position.

CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

Notice: Do not use alcohol based cleaners on this product





**Operating Instructions for the
BIONIX Butterfly Breast Board- Timo &
Silverman**
Product #: RT-5526

WARRANTY

The Bionix Butterfly Breast Board – Timo & Silverman is warranted to be free from defects in workmanship and materials, under normal use, for a period of three years from the date of original purchase from Bionix. The sole obligation of Bionix under this warranty shall be, at the company's option, to replace the product or part with a comparable product or part. To register your product visit: http://www.bionixrt.com/RT_Surveys/WarrantyForm.aspx

Bionix Radiation Therapy
5154 Enterprise Blvd.
Toledo, OH 43612
Toll Free: 800-624-6649
Fax: 800-455-5678
www.BionixRT.com

European Authorized Representative:
Obelis s.a.
34, Av. De Tervuren, bte 44
Brussels, Belgium B-1040
Tel: +32 2 732 59 54

