

The Bionix Prone Breast Hand Rest is intended to be used for the positioning and re-positioning of patients undergoing treatments or imaging.

Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

## COMPONENTS


- A: Face Cushion Indexing Assembly
- B: Indexing Hand Rest
- C: Hand Posts

Face Cushion and Arm Rest Cushion pieces are previously owned and not included in the packaged device. They are included in the assembly diagram as reference points.

The Face Cushion Pedestal is listed as a reference point as well and is a part of the Face Cushion Indexing Assembly.

## ASSEMBLY DIAGRAM

Diagram coloring used to demonstrate component differentiation. May not be actual product color.

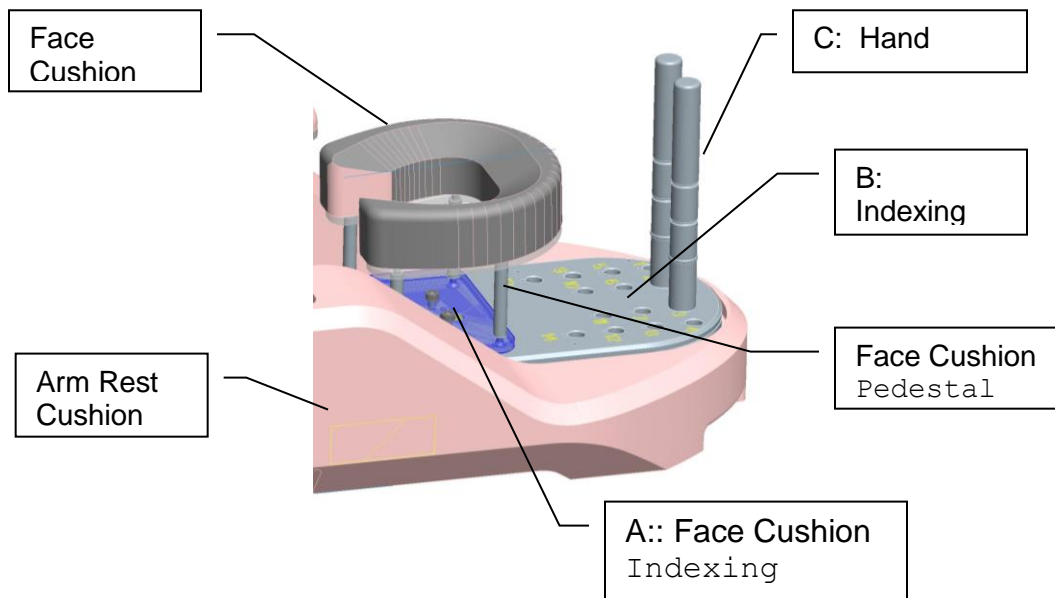

## REPLACEMENT INSTRUCTIONS

When replacing the original face cushion indexing assembly with the new face cushion indexing assembly including the Indexing Hand Rest option, please follow the instructions below.

1. Remove the original face cushion from the Velcro fasteners and set aside temporarily.
2. Carefully remove the entire acrylic plating and face cushion pedestal from the arm rest cushion.
3. Take the Indexing Hand Rest and align the Velcro currently on the arm rest cushion with the Velcro on the Indexing Hand Rest and press down firmly.
4. Place the original face cushion on to the pedestal of the new head rest.

## OPERATING INSTRUCTIONS

After proper positioning of the patient has been achieved, record the location of each hand location relative to which indexing hole the hand posts are in.

## CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

**Notice: Do not use alcohol based cleaners on this product.**

Bionix Radiation Therapy  
5154 Enterprise Blvd.  
Toledo, OH 43612  
Toll Free: 800-624-6649  
Fax: 800-455-5678  
[www.BionixRT.com](http://www.BionixRT.com)

European Authorized Representative:  
**CE** Obelis s.a.  
34, Av. De Tervuren, bte 44  
Brussels, Belgium B-1040  
Tel: +32 2732 59-54