

The Bionix Carbon Fiber Butterfly Board is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.


Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

COMPONENTS

A:: O-Ring
 B:: Locating Screws

C:: Attachment Pegs
 D:: Butterfly Breast Board

ASSEMBLY DIAGRAM

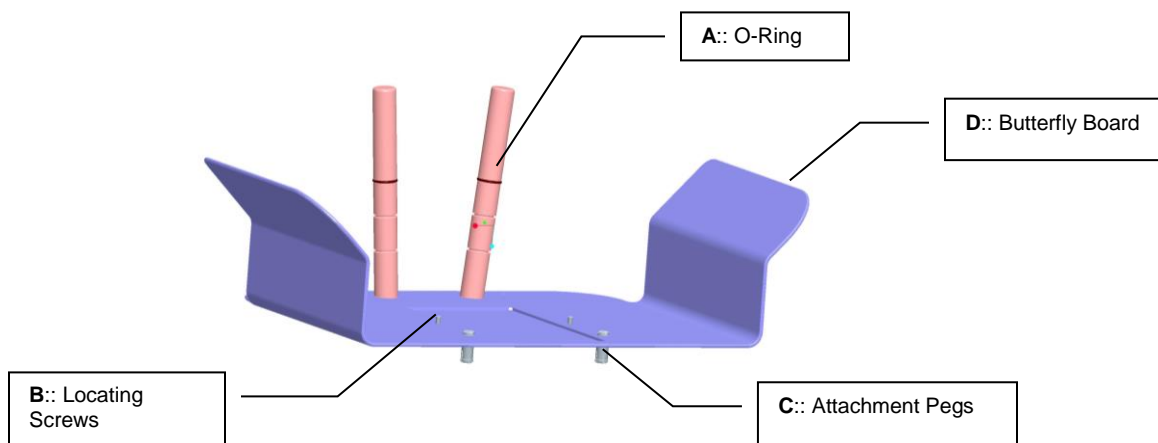

OPERATING INSTRUCTIONS

Note: The Silverman Head Cups must be punched to be used with the Locating Screws

1. Position the Butterfly Breast Board (D) on the treatment table.
2. Position Punched Silverman head cup on the Butterfly Board (D).
3. Position the patient on the Butterfly Board (D).
4. Move the O-rings (A) on the hand posts to the desired position.
5. **(Optional): For attaching to a Bionix Breast Board:** Remove the 2 attachment pegs (C) from the plastic pouch. Remove the screws from the pegs and place each peg into each of the larger holes located on the Butterfly Board (D). The attachment pegs (C) should be attached from the underside of the board (see Assembly Diagram). Secure the pegs to the Butterfly Board (D) by attaching the screws that accompany the pegs. By adding these attachment pegs (C) to the Butterfly Board (D), you can attach the Butterfly Board (D) to any Bionix Breast Board.

Warning: If you plan on using a SecureVac cushion with the Butterfly Board, it is recommended that you first remove the two locating



screws (B) (located near the center of the Butterfly Board) so they do not puncture the SecureVac cushion.

CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

Notice: Do not use alcohol based cleaners on this product.

WARRANTY

The Bionix SecureVac™ Vacuum Cushions is warranted to be free from defects in workmanship and materials, under normal use, for a period of three years from the date of original purchase from Bionix. The sole obligation of Bionix under this warranty shall be, at the company's option, to replace the product or part with a comparable product or part. To register your product visit: http://www.bionixrt.com/RT_Surveys/WarrantyForm.aspx

Bionix Radiation Therapy
5154 Enterprise Blvd.
Toledo, OH 43612
Toll Free: 800-624-6649
Fax: 800-455-5678
www.BionixRT.com

European Authorized Representative:
 Obelis s.a.
34, Av. De Tervuren, bte 44
Brussels, Belgium B-1040
Tel: +32 2 732 59-54